

KaVo Dental Maintenance Guide

Maintaining your KaVo E-Type Low-Speed Attachments

Step 1

Remove the attachment from the motor coupler.




Step 2

Remove bur. (close chuck on straight attachments)



Step 3

Scrub handpiece under running water with a soft sponge to remove external debris.

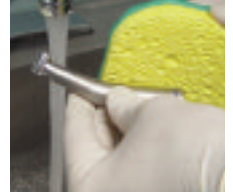

Do not spray, wipe, or immerse in disinfectants or alcohol! Do not combine disinfection with sterilization.

Manual Procedure

Step 4

Remove head (except 25LN, 25LHA, 25LPA). For straight nose cones and off angles make sure the chuck collars are in the closed position without bur before spraying.




Step 5

Place attachment on the KaVo Spray with Quick Nozzle, cover with paper towel and spray thoroughly for 3 seconds. **Other maintenance fluids are not recommended.** If a large amount of debris appears, repeat the process. *Step 5 can be omitted by using the KaVo QUATTROcare automatic maintenance system.*



Step 6

Fit the attachment (without head or bur) onto the motor, allow it to run for 20 seconds **with head down** and wipe off excess lubricant. *Step 6 can be omitted by using the KaVo QUATTROcare automatic maintenance system.*

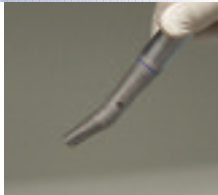

Step 7

Do not re-assemble head to attachment prior to sterilization. Bag and sterilize Low-Speed Attachments without head or bur up to maximum 135°C (275°F) in an autoclave or chemi-clave.



OR

Automatic Procedure

QUATTROcare

Maintaining with the KaVo QUATTROcare is as easy as the following:

Position the instruments...



... close the door ...

... press Start and everything is DONE.

...Go to Step 7



Nozzle



Incorrect
MULTiflex Nozzle

Correct
nozzle use
is very
critical



Correct
Quick Nozzle
#0411.9931

Attention!

KaVo INTRA Low-Speed attachments must be lubricated **BEFORE** every sterilization cycle with KaVo Spray.

— Do not spray after sterilization.