

Maintenance

KaVo MULTiflex Air High-Speed Handpieces


Step 1

Remove the MULTiflex handpiece from the MULTiflex coupler.



Step 2

Remove bur.

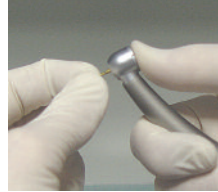

Step 3

Scrub handpiece under running water with a soft sponge to remove external debris.



Do not spray, wipe, or immerse in disinfectants or alcohol! Do not combine disinfection with sterilization.

Manual Procedure

Step 4

Fit the handpiece on the KaVo Spray with the MULTiflex Nozzle. Cover with paper towel and spray thoroughly for 3 seconds. **Other maintenance fluids are not recommended.** If a large amount of debris appears, repeat the process. *Step 4 can be omitted by using the KaVo QUATTROcare automatic maintenance system.*




Step 5

Fit the handpiece without bur onto the MULTiflex coupler or air station, allow it to run for 20 seconds and wipe off excess lubricant. *Step 5 can be omitted by using the KaVo QUATTROcare automatic maintenance system.*



Step 6

Bag and sterilize MULTiflex High-Speed handpiece without bur up to maximum 135°C (275°F) in an autoclave or chemiclave.

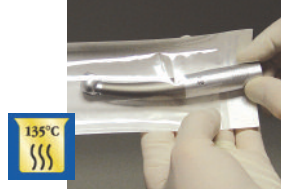

OR

Automatic Procedure

QUATTROcare

Maintaining with the KaVo QUATTROcare is as easy as the following:
Position the instruments...




... close the door ...

... press Start and everything is DONE.
...Go to Step 6



Step 7

The MULTiflex coupler cannot be sterilized.

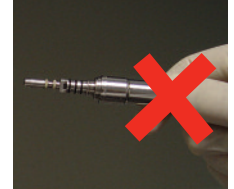

Nozzle



Correct nozzle use is very critical



Attention!

KaVo MULTiflex High-Speed Handpieces must be lubricated **BEFORE** every sterilization cycle with KaVo Spray.
— Do not spray after sterilization.